### XenoSure® Biologic Pledgets

Instructions for use - English

### XenoSure® Biologische Pledgets

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### Compresses biologiques XenoSure®

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### XenoSure® Pledget biologici

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### Parches biológicos XenoSure®

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### Compressas biológicas XenoSure®

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(Model Numbers PL0307R10, PL0506R10, PL0510R10, PL0307R25,

PL0506R25, PL0510R25)

Processed bovine pericardial tissue

Instructions for use - English

**Storage**

The XenoSure® Biologic Pledgets should be stored above 0°C (32°F). Avoid locations where extreme temperature fluctuations may occur; for example, near steam or hot water pipes, air conditioning ducts, or in direct sunlight. REFRIGERATION IS NOT REQUIRED. FREEZING WILL SERIOUSLY DAMAGE THE XENOSURE BIOLOGIC PLEDGET AND RENDER IT UNFIT FOR USE.

**Device Description**

The XenoSure® Biologic Pledgets consist of pieces of bovine pericardial tissue that has been selected for minimal tissue blemishes. The tissue is treated with a glutaraldehyde process which crosslinks the collagen fibers and minimizes antigenicity. The XenoSure Biologic Pledge is liquid chemical sterilized and packaged in a plastic jar containing sterile glutaraldehyde storage solution.

The XenoSure Biologic Pledgets come in the following sizes:

Model		Size (cm)	Shape
PL0307R10	PL0307R25	0.3 cm x 0.7 cm	Rectangular
PL0506R10	PL0506R25	0.5 cm x 0.6 cm	Rectangular
PL0510R10	PL0510R25	0.5 cm x 1.0 cm	Rectangular

**Indications For Use**

The XenoSure Biologic Pledgets are intended for use as a surgical pledge material to secure and support sutures and to aid in suture buttressing during general, vascular and cardiac surgery.

**Contraindications**

None known.

**Potential Complications**

- Restenosis
- Pseudoaneurysm formation
- Infection
- Thrombosis
- Calcification
- Fibrosis
- Vessel occlusion
- Dilatation
- Myocardial infarction
- Bleeding
- Stroke
- Death

**Warnings**

The principal complications that have been reported for bovine pericardial tissue are fibrosis and infection. These complications are observed only in a small minority of patients after implantation of the bovine pericardial tissue.

**Precautions**

All persons responsible for the handling and preparation of the XenoSure Biologic Pledgets must exercise utmost care to avoid damage to the XenoSure Biologic Pledge tissue.

- FOR SINGLE USE ONLY. Do not reuse, reprocess, or resterilize. Reuse, reprocessing, and/or resterilization of the device and/or failure could cause patient injury, illness or death. Any unused XenoSure Biologic Pledgets must be discarded. Note product "Use By" date.
- INSPECT sealed sterile package before opening. If seal is broken, contents may not be sterile and may cause infection in the patient. DO NOT USE. Do not discard the product. Please contact your distributor for further instructions.
- DO NOT expose the device to temperatures below 0°C (32°F). FREEZING WILL SERIOUSLY DAMAGE THE XENOSURE BIOLOGIC PLEDGET AND RENDER IT UNFIT FOR USE. DO NOT STORE UNDER REFRIGERATION.
- RINSE the device according to the "RINSE PROCEDURE" section of this booklet before using. The XenoSure Biologic Pledge storage solution contains glutaraldehyde and may cause irritation of skin, eyes, nose and

- throat. DO NOT BREATHE STORAGE SOLUTION VAPOR. Avoid prolonged skin contact and immediately flush area with water. In case of contact with eyes, seek medical assistance immediately. The liquid chemical storage solution should be disposed according to hospital procedure.
- DO NOT handle the XenoSure Biologic Pledgets with traumatic instruments. This may damage the device.
  - DO NOT use any XenoSure Biologic Pledged that has been damaged. Device integrity may be compromised.
  - DO NOT attempt to repair the XenoSure Biologic Pledgets. Should damage to the XenoSure Biologic Pledged occur before implantation, replace the XenoSure Biologic Pledged.
  - DO NOT resterilize. Unused sections should be considered non-sterile and discarded.
  - DO NOT expose the XenoSure Biologic Pledgets to steam, ethylene oxide, chemical or radiation (gamma/electron beam) sterilization. Damage may result!
  - DO NOT use cutting suture needles or cutting point armed sutures. This may damage the device.
  - DO NOT allow the tissue to dry out during handling.
  - DO NOT use if the device is beyond the expiration date.

#### **Adverse Effects**

Improper functioning of an implanted XenoSure Biologic Pledged produces symptoms identical to symptoms that arise from deficiencies in the natural organ. It is the responsibility of the implanting surgeon to inform the patient of the symptoms that indicate improper functioning of the XenoSure Biologic Pledged.

1. Glutaraldehyde-treated tissue may be subject to late attack by the immune system with subsequent tissue deterioration. The benefits of use of the XenoSure Biologic Pledgets must be weighed against the possible risk of late tissue deterioration.
2. Residual glutaraldehyde presents a risk of toxicological effects. Completing the appropriate rinsing procedure as listed within the IFU reduces the residual glutaraldehyde on the pledged to an acceptable level and therefore significantly reduces the risk of acute toxicological effects. Review of published literature has not resulted in an established safe limit for glutaraldehyde exposure when implanted within the vasculature. The risks increase when implanting large amounts of glutaraldehyde treated tissue or within patients with less mass. The benefits of use of the XenoSure Biologic Pledged must be weighed against the possible risk of toxicological effects.
3. Animal studies with bovine pericardium have reported calcification and histological signs of deterioration as an adverse reaction. Findings include phagocytosis with accompanying chronic inflammatory infiltrate at the interface between bovine pericardium and surrounding host tissue with focal degradation of implant collagen consistent with host vs. graft reaction.

#### **How Supplied**

The XenoSure Biologic Pledgets are provided sterile and non-pyrogenic in a sealed container; DO NOT RESTERILIZE. The pledges are stored in a phosphate buffered sterile physiological saline solution containing 0.2% glutaraldehyde. Sterility is assured if the package is unopened and has an undamaged seal. Unused pledges should be considered non-sterile and discarded.

#### **Directions For Use**

Choose the required XenoSure Biologic Pledged model as appropriate for the type of procedure being performed. The XenoSure Biologic Pledged can be cut to a size appropriate for a given repair. XenoSure Biologic Pledged is for SINGLE USE ONLY.

#### **Pledged Preparation**

Surgical gloves must be thoroughly washed to remove all powder residues before handling the XenoSur Biologic Pledgets.  
Examine the information of the jar label to verify selection of the correct XenoSure Biologic Pledged size. Carefully inspect the entire container and tamper-evident seal for damage.  
DO NOT USE THE XENOSURE BIOLOGIC PLEDGETS IF THE JAR IS DAMAGED OR IF THE SEAL IS BROKEN. Do not discard the product. Please contact your distributor for further instructions.

#### **Rinse Procedure**

The appropriate rinse procedure must be followed in order to reduce patients exposure to residual glutaraldehyde.  
Remove the tamper-evident outer plastic seal and unscrew the jar cap. The

contents of the jar are sterile and must be handled aseptically to prevent contamination. The outside of the jar is not sterile and must not enter the sterile field.

From the jar, remove the XenoSure Biologic Pledgets by grasping the basket post with gloved hands or sterile forceps, allowing excess storage solution to drain into the jar.

Once removed from the container, submerge the XenoSure Biologic Pledged basket in the sterile physiological saline solution. Using the same forceps, gently agitate the XenoSure Biologic Pledged in the basin. Allow the XenoSure Biologic Pledged to remain in the rinse basin until required by the surgeon.

At the surgeon's discretion the rinse solution may contain bacitracin (500 U/mL) or cephalaxin (10 mg/mL), as testing has shown that the XenoSure® bovine pericardial pledge material is not adversely affected by treatment with those antibiotics. The effects of other antibiotics or the long term effects of these antibiotics on the XenoSure® bovine pericardial pledge material have not been tested. Use antibiotics only as indicated by the antibiotics manufacturer.

The pledges require a rinse time of 2 minutes in 500ml of saline.

#### **Implantation**

During implantation, irrigate the XenoSure Biologic Pledged tissue frequently with sterile physiologic saline to prevent drying.

#### **Surgical Technique**

It is beyond the scope of this Instructions for Use booklet to instruct the surgeon in specific repair procedures. LeMaitre Vascular, Inc. assumes that any surgeon performing the above operations has received adequate training and is thoroughly familiar with the pertinent scientific literature.

#### **Clinical Experience**

Bovine pericardial tissue has been successfully used in intracardiac repair procedures for the past 25 years with excellent clinical results. Published data on such use may be found in medical literature.

At this time, the long term durability of fixed bovine pericardial tissue is unknown. The clinical advantages in using this material along with its known characteristics must be weighed against its current undetermined long-term durability.

#### **Limited Product Warranty; Limitation of Remedies**

LeMaitre Vascular, Inc. warrants that reasonable care has been used in the manufacture of this device. Except as explicitly provided herein, LEMAITRE VASCULAR (AS USED IN THIS SECTION, SUCH TERM INCLUDES LEMAITRE VASCULAR, INC., ITS AFFILIATES, AND THEIR RESPECTIVE EMPLOYEES, OFFICERS, DIRECTORS, MANAGERS, AND AGENTS) MAKES NO EXPRESS OR IMPLIED WARRANTIES WITH RESPECT TO THIS DEVICE, WHETHER ARISING BY OPERATION OF LAW OR OTHERWISE (INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE) AND HEREBY DISCLAIMS THE SAME. LeMaitre Vascular makes no representation regarding the suitability for any particular treatment in which this device is used, which determination is the sole responsibility of the purchaser. This limited warranty does not apply to the extent of any abuse or misuse of, or failure to properly store, this device by the purchaser or any third party. The sole remedy for a breach of this limited warranty shall be replacement of, or refund of the purchase price for, this device (at LeMaitre Vascular's sole option) following the purchaser's return of the device to LeMaitre Vascular. This warranty shall terminate on the expiration date for this device.

IN NO EVENT SHALL LEMAITRE VASCULAR BE LIABLE FOR ANY DIRECT, INDIRECT, CONSEQUENTIAL, SPECIAL, PUNITIVE, OR EXEMPLARY DAMAGES. IN NO EVENT WILL THE AGGREGATE LIABILITY OF LEMAITRE VASCULAR WITH RESPECT TO THIS DEVICE, HOWEVER ARISING, UNDER ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, TORT, STRICT LIABILITY, OR OTHERWISE, EXCEED ONE THOUSAND DOLLARS (US\$1,000), REGARDLESS OF WHETHER LEMAITRE VASCULAR HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH LOSS, AND NOTWITHSTANDING THE FAILURE OF THE ESSENTIAL PURPOSE OF ANY REMEDY. THESE LIMITATIONS APPLY TO ANY THIRD-PARTY CLAIMS.

A revision or issue date for these instructions is included on the back page of these Instructions for Use for the user's information. If twenty-four (24) months has elapsed between this date and product use, the user should contact LeMaitre Vascular to see if additional product information is available.

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LeMaitre and XenoSure are registered trademarks of LeMaitre Vascular, Inc.
©2018 LeMaitre Vascular, Inc. All rights reserved.
LeMaitre Vascular LLC
LeMaitre Vascular GmbH